



## Content

- 1 Night and Day calendar**
- 1 Adaptor (battery eliminator)**

## Instructions for use

- 1. Connect one end of the adaptor to an outlet, and the other end to the calendar**
- 2. After 1-2 seconds, day and date and time of the day will be shown on the display Morning (from 6 a.m. to 12), Afternoon (from 12 to 6 p.m.), Evening (from 6 p.m. to 10 p.m.), and Night (from 10.p.m. to 6 a.m.). The calendar is set for winter time.**

The text will disappear in case of power failure, but will appear when the current is reconnected. This function is dependent upon a battery inside the calendar. If the calendar is disconnected for a longer period (about 1 year), the battery must be replaced in order to maintain memory of day and date.

## Adaptor

Only the adaptor provided with the calendar must be used. The sticker with the Forget-me-not trade mark may be fastened to the adaptor in order to remember this.

## How to set the calendar

The calendar is correctly set by the producer, and resetting is normally not needed. If resetting is needed, please follow the instructions below.

1. Press the two buttons at the back simultaneously. The calendar will be in a setting mode, as can be seen from the blinking black cursor. The information shown on the display can now be changed as described below.
2. To move the cursor to the desired position, press the right hand button (seen from the front).
3. Press the left hand button to set the correct information (day, month, year, time).

## Back-up battery (Lithium – ion, CR 2032)

If the day or date is wrong the back-up battery may need to be replaced. To check the battery set the calendar to the correct day and date (see instruction above). Disconnect and then reconnect the power. If the calendar shows incorrect day or date the battery must be replaced as follows:

First, disconnect the power. Then separate calendar cover and calendar by pressing down the cover base and slide the calendar out. Carefully detach the silver-coloured film and remove the electronic card. To remove the battery, press it towards one of the white plastic clips. To insert the new battery press until it clicks.

Forget-me-not AS, P.O. Box 7167 Majorstua, 0307 Oslo, Norway

Tel (int.): +(47) 95 19 98 95

Fax (int): + (47) 22 14 37 60

E-mail: [post@forglemmegei.no](mailto:post@forglemmegei.no)

Home page: [www.forget-me-not.no](http://www.forget-me-not.no)